## Headrest Operation & Installation

EN | FR

Look closer. See further.

 HAAG-STREIT  
RELIANCE

## SERIAL NUMBER

---

For your future reference, mark the serial number in the space provided.

**Haag-Streit USA Inc.**

3535 Kings Mills Road  
Mason, Ohio 45040-2303

(800) 787-5426

[www.haag-streit-usa.com](http://www.haag-streit-usa.com)

**MADE IN USA**

QUALITY MANAGEMENT  
SYSTEM  
REGISTERED TO  
ISO 13485:2016

**TABLE OF CONTENTS**

**IMPORTANT INFORMATION .....4**  
    **SAFETY SYMBOLS .....4**  
    **TRANSPORTATION / STORAGE CONDITIONS .....4**  
**1. INTRODUCTION .....4**  
**2. INSTALLATION .....5**  
    **2.1. Unpacking and Attaching Headrest .....5**  
        **2.1.2. Operation – No. 18, 22 Headrest ...6**  
        **2.1.3. Adjustment – No. 18, 22 Headrest**  
            **.....6**  
        **2.1.4. Operation – No. 10 Headrest .....6**

**CLEANING AND MAINTENANCE .....7**  
**LIMITED WARRANTY .....9**

## IMPORTANT INFORMATION

### SAFETY SYMBOLS

Whenever you see the symbols shown below, heed their instructions! Always follow safe operating and maintenance practices.

**“WARNING” – THE WARNING SYMBOL IDENTIFIES SPECIAL INSTRUCTIONS OR PROCEDURES WHICH, IF NOT CORRECTLY FOLLOWED, COULD RESULT IN PERSONAL INJURY.**

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**“CAUTION” – THE CAUTION SYMBOL IDENTIFIES SPECIAL INSTRUCTIONS OR PROCEDURES WHICH, IF NOT STRICTLY OBSERVED, COULD RESULT IN DAMAGE TO OR DESTRUCTION OF EQUIPMENT.**

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**“NOTE”** – Note indicates points of particular or additional information.

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Not To Be Used as a Handle

## TRANSPORTATION / STORAGE CONDITIONS

Temperature: Operating:	59 to 104° F 15 to 40° C
Storage:	-22 to +140° F -30 to +60° C
Relative Humidity: Operating:	5%RH to 95%RH without condensation
Storage:	10%RH to 100%RH, including condensation
Atmospheric Pressure:	
Operating:	700hPa – 1060hPa
Storage:	500hPa – 1060 hPa

### 1. INTRODUCTION

- 1.1. This manual contains information applicable only to the Reliance® Headrest Models No. 18, 22, and 10. Should you have a different model, please disregard any information that does not apply.
- 1.2. Should your product not perform properly, or if the product is in need of repair, or if you have any questions concerning the use and care of any Haag-Streit product, contact the Haag-Streit Distributor, where you purchased this product or contact the Technical Service Department, Haag-Streit USA, Inc., 3535 Kings Mills Road, Mason, Ohio 45040-2303, or call 800-787-5426.

**NOTE:** Always have the model number and serial number available before contacting Haag-Streit or your authorized Haag-Streit Distributor.

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## 2. INSTALLATION

**WARNING: INSTALLATION & REPAIRS MAY ONLY BE PERFORMED BY TRAINED TECHNICAL PERSONNEL.**

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
### 2.1. Unpacking and Attaching Headrest

**NOTE:** Intended use of the Headrest is designed for comfort and support during normal seated use only.

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**2.1.1.** Remove the headrest from its carton and inspect for any damage from transportation before proceeding. Mount headrest into position onto the chair back by sliding the headrest into the Headrest Bracket. Using a 5/32" hex key wrench, tighten the cap screw to clamp the headrest securely into its mounting bracket. (See Figure 1).

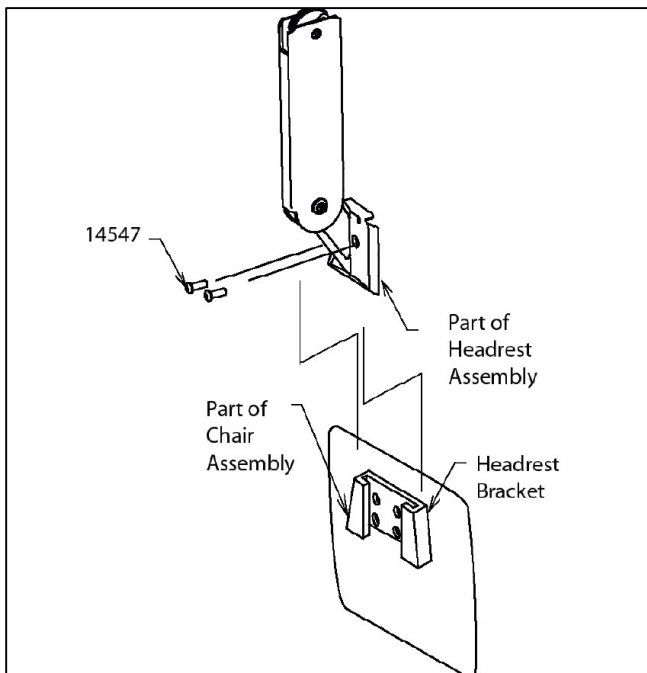**FIGURE 1  
NO. 18, 22 HEADREST**

**WARNING: IF THE NO. 18 OR 22 HEADREST RECEIVES AN IMPACT OR STRIKES AN OBJECT WITH FORCE CONTACT HAAG-STREIT TECH SERVICE IMMEDIATELY.**

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**WARNING: NEVER USE THE HEADREST AS A HANDLE WHEN REMOVING THE CHAIR FROM THE SKID OR DURING TRANSPORT TO ITS FINAL LOCATION.**

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**WARNING: NEVER OPERATE DEFECTIVE EQUIPMENT. CONTACT HAAG-STREIT TECH SERVICE IMMEDIATELY.**

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**WARNING: ONLY TRAINED/QUALIFIED PERSONNEL SHOULD OPERATE HEADREST.**

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### 2.1.2. Operation – No. 18, 22 Headrest

2.1.2.1. The No. 18, 22 headrest is designed so it can be operated with just one hand. To unlock, squeeze the clear plastic actuator into lock housing. This will allow both pivots of the articulating arm to rotate so the headrest can be moved to a new desired position. Once in the new position, release the actuator. Make sure the actuator expands to its original position fully exposing black dot on handle to insure a positive lock. The headrest pad is free to rotate at all times (See Figure 2).

### 2.1.3. Adjustment – No. 18, 22 Headrest


2.1.3.1. There are three adjustments provided on the No. 18, 22 headrest. The first is the tension on the pad rotation. By tightening the two hex acorn nuts on the pad pivots with a 3/8" open end wrench, the tension will be increased.

2.1.3.2. To adjust the drag on the pivots of the articulating arm, loosen the set screw in the center of the pivot (Figure 2, item 1) with a 3/32" hexagon key wrench. Next, tighten the pivot screw on the other side of the pivot, with a 1/8" hexagon key wrench. Now, retighten the set screw to retain the pivot screw at the new adjustment. Either pivot may be adjusted.

2.1.3.3. The third adjustment is for proper clearance of the block dot to indicate that the headrest is properly locked. If the dot does not fully clear the lock housing when the headrest is lock securely, turn the set screw located at the top of the lock housing (Figure 2, item 2) counterclockwise until the dot on the actuator is fully exposed.

### 2.1.4. Operation – No. 10 Headrest

2.1.4.1. The No. 10 headrest has a fixed, rigid upholstered backboard which supports a sliding, center pillow section. For patient comfort simply move this pillow to a desired position.

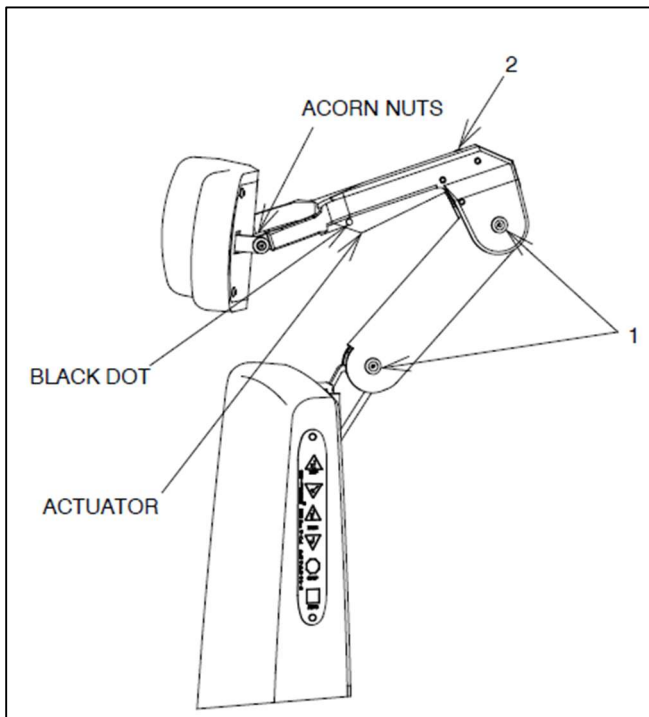**FIGURE 2**  
**NO. 18, 22 HEADREST**

## CLEANING AND MAINTENANCE

### Cleaning the Upholstery

When cleaning upholstery, it is suggested that you use all recommended commercial cleaning agents in an inconspicuous spot to check for potential damage to the upholstery material before applying to the entire area to be cleaned.

#### Day to Day Soil:

Remove ordinary dirt and smudges with a mild soap and warm water solution. Dry with a soft, lint-free cloth or towel. For more difficult stains, use of a stronger detergent is recommended; however, follow the detergent manufacturer's instructions closely.

#### Disinfecting Material:

In order to disinfect, use a 1:9 solution of bleach and water (1 part bleach to 9 parts water). Wipe surface with damp cloth then rinse with water immediately, using a damp cloth.

#### Special Cleaning Problems:

The following methods are recommended to clean stains on PreFixx-protected vinyl upholstery. Most stains will be removed when these cleaning agents are used in the following order.

**Type 1 Cleaners:** Non-abrasive household cleaners to be used with water and a soft cloth.

- Formula 409® All-Purpose Spray Cleaner.
- Fantastik® Spray Cleaner.
- Other similar household cleaners and bleaches.

**Type 2 Cleaners:** Solvent-type cleaner to be liberally applied with a soft cloth. Dry area with another cloth, rinse with clean water and dry.

- Rubbing alcohol (isopropyl alcohol).
- Lighter fluid (naphtha).

**Type 3 Cleaners:** Strong, active solvent cleaners such as nail polish remover (acetone/water) to be applied with a soft cloth. Stain should be removed with less than six (6) rubs; if stain persists after six rubs, contact

manufacturer. Dry area with another cloth, rinse with clean water and dry.

**CAUTION: THESE SOLVENTS ARE HIGHLY FLAMMABLE. EXERCISE PROPER CARE IN CLEANING AND NOTIFY PERSONNEL IN AREA OF DANGER. WEAR RUBBER GLOVES DURING ALL CLEANING ACTIVITY. USE CAUTION IN CLEANING AROUND BUTTONS, STITCHING AND WOODEN OR OTHER DECORATIVE TRIM, SINCE THESE SOLVENTS COULD SERIOUSLY DAMAGE SUCH AREAS.**

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### Cleaning Painted Surfaces

Painted metal surfaces are covered with a durable powder coat paint which is resistant to scratching and scuffing. Painted surfaces may be cleaned with a clean cloth dampened with mild, soapy water or equivalent household spray cleaner.

### Recommended Infection Control Products

The following Infection Control Products have been tested on the equipment and will not cause damage to upholstery, painted surfaces or plastic covers:

- Cavicide® Hospital Disinfectant.
- Precise® Hospital Foam Cleaner Disinfectant.

**WARNING: REFER TO LOCAL OR HEALTHCARE PROFESSION GUIDELINES FOR SELECTING EFFECTIVE INFECTION CONTROL PRODUCTS.**

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**CAUTION:** FOLLOW MANUFACTURER'S DIRECTIONS FOR CONCENTRATION AND APPLICATION OF DISINFECTING CLEANER. AVOID PROLONGED APPLICATION OF ANY DISINFECTANT/CLEANER PRODUCTS, BECAUSE THEY MAY CAUSE STAINING OR DISCOLORATION OF MATERIAL.

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**NOTE:** Equipment should be cleaned as needed or per local or healthcare profession guidelines.

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**WARNING:** TO AVOID BIOLOGICAL HAZARDS WHEN EQUIPMENT HAS REACHED THE END OF ITS USEFUL LIFE, DISPOSE OF EQUIPMENT PER LOCAL REGULATIONS OR HEALTHCARE PROFESSION GUIDELINES.

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**NOTE:** To ensure a long service life, the device should be cleaned as described. It is also recommend having the device inspected by an authorized service technician annually.

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## LIMITED WARRANTY

The Reliance® product must be used only for the purposes and in the manner described in the literature distributed with the product. The products are warranted against defective materials and workmanship for a period of two (2) years from date of installation. Products or parts thereof will be repaired or replaced as required at Haag-Streit USA, Inc. Such repair or replacement shall be the sole remedy under this warranty. This warranty extends only to the original purchaser from an authorized Haag-Streit dealer and is subject to the following conditions:

1. The warranty card must be completed and returned to Haag-Streit USA, Inc. within two (2) weeks from the date of installation.
2. Installation and servicing of the products must be performed by trained Haag-Streit USA, Inc. equipment dealer service personnel in accordance with the appropriate instruction manual for the products. Warranty will be voided if installation is performed by a non-authorized individual.
3. The products have not sustained breakage or other types of damage due to accident or misuse.
4. This warranty will not apply to Reliance® products which have had the serial number removed, altered or effaced.

**EXCEPT FOR THE EXPRESS WARRANTY SET FORTH ABOVE, HAAG-STREIT DOES NOT GRANT ANY WARRANTIES, EITHER EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE.**

Haag-Streit USA, Inc. neither assumes nor authorizes any person to assume for it, any other liability in connection with the sale and use of its products.

**REMEDIES ARE LIMITED EXCLUSIVELY TO REPAIR OR REPLACEMENT OF PARTS. HAAG-STREIT USA, INC. EXPRESSLY DISCLAIMS LIABILITY FOR INCIDENTAL OR CONSEQUENTIAL DAMAGE RESULTING FROM THE USE OF THE EQUIPMENT.**

Claims covered by this warranty will be honored when presented within one (1) month from discovery of a defect.

**MADE IN USA**

**Haag-Streit USA, Inc  
3535 Kings Mills Road  
Mason, Ohio 45040-2303 USA  
(800) 787-5426  
(Customer / Technical Service)  
(513) 398-3937  
<http://www.haag-streit-usa.com>**

**P/N 27369 REV J  
© Copyright 2015  
Revised 04-2026-10014**